ResMed

Air11™

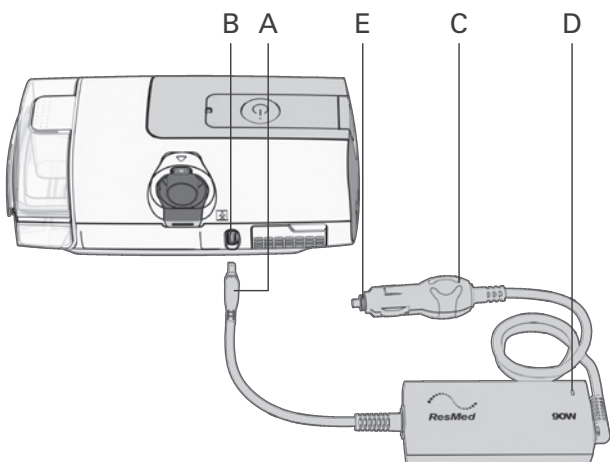
DC/DC Converter 90W

User guide

English | Deutsch | Français | Italiano | Nederlands | Español
Português | Dansk | Íslenska | Svenska | Suomi | Norsk | Český
Eesti | Ελληνικά | Polski | Magyar | Slovenščina | Română
Русский | Türkçe | Български | Hrvatski | Lietuvių k. | Latviski
Français canadien | Español latinoamericano
Português do Brasil | Bahasa Melayu | Bahasa Indonesia

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Indications for use

The Air11™ DC/DC converter is intended to allow operation of the Air11 device from a vehicle auxiliary power outlet (12V or 24V DC power source) in a car, boat, or other vehicle equipped with a suitable battery.

The converter (model no. 390002) is specifically for use with an Air11 device.

WARNING

Read this entire guide before using the device.

Using the converter via a vehicle auxiliary power outlet

Refer to the illustration at the front of this user guide.

1. Position the converter on a level surface, with the ResMed logo facing up.
2. Plug the converter connector **(A)** into the device power inlet/DC input socket **(B)**.
3. Plug the converter adapter plug **(C)** into the vehicle auxiliary power outlet. The light **(D)** will glow green when it powers on. If the battery is low, the light will glow amber. If the battery is too low to operate the device, the converter and light will turn off.

Notes:

- Some vehicles require the ignition key to be turned to the "accessories" position to provide power to the vehicle auxiliary power outlet.
- The power cord is not replaceable.
- If the battery OR car, boat or other vehicle are not in good condition, there may not be enough reserve left in the battery to start the vehicle after use.
- When the converter is not in use, disconnect it from the vehicle auxiliary power outlet to prevent battery discharge.
- The converter must only be used with an Air11 device to ensure proper performance.
- ResMed recommends connecting a deep cycle battery to the converter to power your device, as deep cycle batteries are optimised for frequent heavy discharges.
- If you need help or to report an unexpected issue, contact a ResMed representative, ResMed Technical Services or see ResMed.com.

WARNING

- Only use the converter when connecting to a 12V or 24V DC supply.
- Replace fuse with same type and ratings to avoid risk of fire.
- Do not attempt to modify the converter. There are no user-serviceable parts in the converter.
- Do not plug the power cord into a USB port.

- Make sure the power cord and plug are in good condition and the equipment is not damaged.
- Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended 3.9" (10 cm) separation distance.
- The device should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the device should be observed to verify normal operation in the configuration in which it will be used.

CAUTION

- The converter is intended for use on 12V or 24V negative ground systems (where the negative terminal of the battery is grounded). Using it on a 12V or 24V positive ground system may damage the converter and the vehicle's electrical system. If you are unsure whether you have a negative ground system, consult the car, boat or other vehicle manual.
- Make sure the converter is not covered during use to avoid overheating the converter.
- Make sure the converter is positioned safely, where it will not drop or become a trip hazard.
- The Air11 DC/DC Converter is not intended for use on an aircraft.

Note: For any serious incidents that occur in relation to this device, these should be reported to ResMed and the competent authority in your country.

Cleaning

If necessary, wipe the outside of the converter with a clean, damp cloth.

Changing the fuse

1. Unscrew the fuse holder (E) from the top of the converter adaptor plug (C).
2. Remove the old fuse and insert a replacement fuse of the same type and rating.
3. Re-screw the fuse holder.

Technical specifications

Length (connector to connector)	9.8 ft (3m)
Fuse rating	12A/250V
Fuse type	Slow blow
Input	12V/24V  10A/5A
Output	24 V  3.75A
IEC 60601-1 (Edition 3.1) classification	Class II (double insulation), Ingress protection IP22.
Operating temperature	+41°F to +95°F (+5°C to +35°C)
Operating humidity (relative humidity, non-condensing)	10-95%

Operating altitude	Sea level to 9,870 ft (3010m); (air pressure 1060 to 700hPa)
Storage and transport temperature	-13°F to +158°F (-25°C to +70°C)
Storage and transport humidity (relative humidity, non-condensing)	5-95%
Design life	5 years
Electromagnetic compatibility	Product complies with all applicable electromagnetic compatibility requirements (EMC) according to IEC60601-1-2, for commercial and light industry environments. Portable RF Communications equipment (including peripherals such as antenna cables and external antenna) should be no closer than 3.9" (10cm) to any part of the Air11 system. Otherwise degradation of the performance of this equipment could result. Information regarding the electromagnetic emissions and immunity of this ResMed device can be found on www.resmed.com, on the Products page under Service and Support.

Symbols

 Follow instructions before use.  Indicates a warning or caution.  Manufacturer.  Direct current.  Class II equipment. **IP22** Protected against finger sized objects and against dripping water when tilted up to 15 degrees from specified orientation.  Fuse  Temperature limitation.  Humidity limitation.  Operating altitude.  Atmospheric pressure limitation.  Date of Manufacture  Catalog number.  Serial number.  Batch code.  European Authorized Representative.  Medical device.  Importer.

See symbols glossary at ResMed.com/symbols.

■ Environmental information (EU directive 2012/19/EE Waste Electrical and Electronic Equipment (WEEE)).

The DC/DC Converter should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment.

If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to [ResMed.com/environment](https://www.resmed.com/environment).

Limited warranty

ResMed warrants that your ResMed product shall be free from defects in material and workmanship for a period of 1 year from the date of purchase by the initial consumer. This warranty is not transferable. During the warranty period, if the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This limited warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organisation that has not been expressly authorised by ResMed to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; d) any damage caused by water being spilled on or into the product.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty is in lieu of all other express or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have resulted from the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

Visit [ResMed.com](https://www.resmed.com) for the latest information on ResMed's Limited Warranty.

Further information

If you require additional information on how to setup, use or maintain the Air11 system (including DC/DC Converter 90W), or to report unexpected operation or events, please contact the ResMed Service Centre or your care provider.

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© 2024 ResMed. 398192/2 2024-01
See ResMed.com for other ResMed locations worldwide. Air11 is a trademark and/or registered trademark of the ResMed family of companies. For patent and other intellectual property information, see ResMed.com/ip
ResMed Pty Ltd