RC-PFLEGE® N (Nasen- und Rachenspray)

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RC-PFLEGE® N (nasal and throat spray)

for daily administration in the nose and throat

Increasing numbers of people suffer from dry, irritated mucous membranes of the nose and throat. This is caused by general air pollution, frequent visits to places of low humidity, such as heated or air conditioned rooms, and/or long-term oxygen therapy. These factors – air pollution, low humidity and/or long-term oxygen therapy – dry out the mucous membranes of the nose and throat.

However, the filter function of the nose, i.e. the retention of particles existing in the air (dust, pollen, allergens etc.) can only be optimally fulfilled by a moist nasal mucous membrane with a functioning, healthy, ciliated epithelium. In order to ensure the filter and protective function of our nose, it is important to keep the nasal mucous membrane intact and protect it from drying out.

1. COMPOSITION

RC-Pflege® N is a nasal and throat spray that contains sodium chloride and 2% dexpanthenol in an isotonic, sterile solution without preservatives.

RC-Pflege® N nasal and throat spray is sprayed using a mechanical pump system and therefore contains no irritant propellant gas that can be inhaled. The special spray system contains a sterile filter which keeps the solution sterile, even after repeated use. **RC-Pflege® N** nasal and throat spray can therefore be used until the date specified on the package. Preservatives, which can trigger allergic reactions in many people, are not necessary thanks to the sterile filter.

2. INDICATIONS

RC-Pflege® N is administered:

- to **moisten** dry mucous membranes of the nose and throat caused by dry air in heated or air-conditioned rooms, or in cases of long-term oxygen therapy
- to **clean** the nostrils of dust, pollen, allergens or dry crusts
- as a **concomitant therapy** to treat colds, hay fever and perennial allergic rhinitis or
- as a supplement to conventional cold remedies (e.g. decongestant nasal drops)

There are no reservations regarding use during pregnancy and breastfeeding. **RC-Pflege® N** nasal and throat spray **with a sterile filter, without preservatives** is suitable for adults and children over the age of 1 year.

3. ADMINISTRATION AND DOSAGE



Spray 1-2 times in each nostril several times a day **to moisten the nasal mucous membrane**.

A few sprays in the throat are required several times a day **to moisten the nasal mucous membrane**.

Spray 2-3 times in each nostril several times a day **to clean the nose**.

Remove the protective cap before administration and pump it a few times until a smooth spray mist comes out **RC-Pflege® N** nasal and throat spray is now ready for administration.

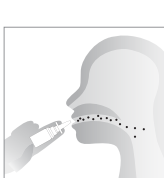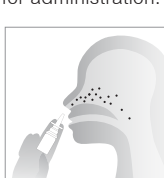The spray bottle should be kept as vertical as possible when spraying so that the suction pipe projects into the liquid. If the spray nozzle stops working properly, the pump mechanism should operated several times in a vertical position until the spray mist comes out again. Wipe off the nose adapter after administration and re-seal the spray with the protective cap.

4. WARNINGS

RC-Pflege® N nasal and throat spray should not be administered if you are hypersensitive (allergic) to any of the ingredients of **RC-Pflege® N** nasal and throat spray.

In cases of an injured nose or following surgery in the area of the nose, **RC-Pflege® N** nasal and throat spray should only be administered after consulting with your doctor.

RC-Pflege® N nasal and throat spray is a medical device! Seek medical advice if symptoms do not improve after several days of treatment with **RC-Pflege® N**. In the event of significant crust formation or inflammation, seek immediate medical attention. **RC-Pflege® N** nasal and throat spray is safe to administer and can therefore be used for extended periods (up to 30 days) as required.

For hygiene reasons and to prevent the transmission of pathogens, a spray bottle should be used by **one** person only.

RC-Pflege® N nasal and throat spray is produced without preservatives and filled with a special application system under sterile conditions.

If the container or application system has been tampered with, do not use **RC-Pflege® N** nasal and throat spray as safe administration can no longer be guaranteed.

When administering at the same time as another locally acting cold remedy, **RC-Pflege® N** nasal and throat spray should be administered **first**.

5. PACKAGE SIZE

Bottle with 20 ml sterile solution, including spray attachment, 1 spray = 0.14 ml solution

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Produkte GmbH
R. Cegla GmbH & Co. KG