SomnoShirt®

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SomnoShirt®

Medical device for the treatment of habitual snoring in the supine position and position-dependent mild and moderate obstructive sleep apnea.

Instructions (Read carefully before use)

Description: SomnoShirt® is worn during sleep. It encourages you to sleep on your side and prevents you from turning onto your back during the night. In contrast to sleeping on your side, sleeping on your back can promote snoring and nocturnal breathing interruptions. The pocket in the back area contains either an ultra-light rigid foam roll (standard version) or a mouth-inflatable air cushion (comfort version) for positioning.

Indications:

- Habitual, position-dependent snoring
- Position-dependent mild and moderate obstructive sleep apnea (after medical consultation)

Contraindications:

- (Severe) spinal complaints
- Convalescence period after back or intervertebral disc surgery
- Positional vertigo
- Position-independent sleep apnea
- Age under 18 years

Warnings:

- Snoring can be a symptom of breathing arrest. The cause should therefore be clarified by a doctor before starting treatment.
- Particularly in patients with nocturnal breathing arrests due to obstructive sleep apnea, the therapeutic effect of the positioning therapy should be checked by appropriate diagnostic measures quite soon after the beginning of treatment.
- In the case of obstructive sleep apnea, existing heart disease, significant overweight or back problems, SomnoShirt® should only be used under medical supervision.

Care instructions: Wash the shirt at 40°C in the washing machine, spin at low speed and hang to dry. The roll or air cushion must be removed before washing the shirt.

Material of the shirt: 100% cotton

Medical device

Art. No.: 07-X

Warnings

Follow instructions for use

Date of information: 03.02.2025
Doc. No.: UI.SSH.2025.GE.PDF

Manufacturer:

Tomed GmbH • Medical Products

Im Rapsfeld 57 • 50933 Köln (Cologne) • Germany

Tel.: +49 221 17067206 • Fax: +49 221 17067207

Email: info@tomed.com • www.tomed.com

The product meets all the provisions of the
Medical Device Regulation 2017/745 which apply to it.